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[English] Instrument Required Disassembly for Cleaning

BENCOX Total Hip System

Part No.	Part Name	Disassembly Instruction
T9509	MPR Acetabular Reamer System - Offset reamer handle	Disassemble by push and twist the collar of outer part to extract the interior part which is divided into two halves and interior drive chain.

LOSPA Total Knee System

Part Number	Instrument name	Disassembly Instructions
01.63.108	Multi-proximal tibial resector RT	Disassemble by turning all parts including alignment rod, indicator, locking knob counter-clock-wise.
01.63.109	Multi-proximal tibial resector LT	Disassemble by turning all parts including alignment rod, indicator, locking knob counter-clock-wise.
01.61.121	EM Alignment Rod 800mm	Disassemble two EM rod from O-ring assembly mechanism.
KA.LTC.1L11	Multi-proximal tibial resector LT(ACF type)	Disassemble by turning all parts including alignment rod, indicator, locking knob counter-clock-wise.
KA.LTC.1L12	Multi-proximal tibial resector RT(ACF type)	Disassemble by turning all parts including alignment rod, indicator, locking knob counter-clock-wise.
KB.ST0.0002	Tibial Implant Jig	Disassemble parts from instrument body.
KA.SF0.0004	Femoral extractor	Disassemble POMC tip by pulling it out to separate it from instrument body.
KA.SF0.0003	Femoral holder	Disassemble POMC tip by pulling it out to separate it from instrument body.
KA.SG0.0005	Torque wrench	Disassemble handle from instrument body.
KA.DFA.0003	AP Sizer	Disassemble stylus assy from instrument body.